

CAPIJECT® Safety Lancets

A legacy of proven design and trusted performance

■ Unmatched design ■ Saves time ■ Greater accuracy ■ Promotes safety

- Ergonomically shaped for comfortable use
- Promotes confident operation and handling
- Single-hand, one-step operation
- Reduced training time
- No special staging, ready to use right out of the box
- Correct product selection is easier with color codes
- Top mounted activation button maximizes visibility for precise placement
- Specifically designed for single use only, preventing reuse
- Automatic retraction of lancet minimizes exposure of sharp and possible contamination



Packaging: 200 Lancets per box / 10 boxes per case – 2,000 Lancets

Product Code	Lancet Color	Width/ Gauge	Depth	Blood Volume
200101	Pink	1.5mm	1.0mm	Low Blood flow
200102	Green	1.5mm	1.5mm	Medium Blood flow
200103	Blue	1.5mm	2.0mm	High Blood flow
200104	Orange	23G	2.25mm	Single Blood Drop
200105	Purple	28G	1.25mm	Single Blood Drop


Call your Terumo representative at:

1.800.888.3786

or visit www.TerumoTMP.com and click on
Find Your Local Terumo Representative


The Perfect Pairing of Ease and Precision

Capiject® Safety Lancets Instructions For Use

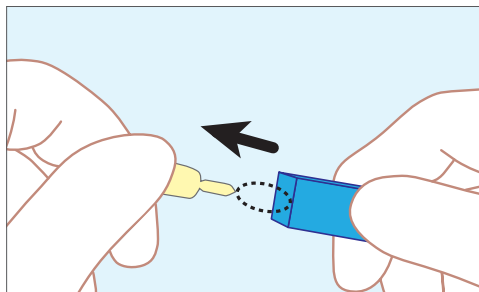


Step 1

After cleaning puncture site, twist off protective tab for needle or pull off for blade and place in disposal.

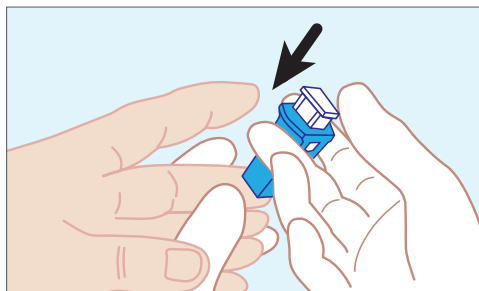


Step 2

Position lancet properly on puncture site as illustrated.

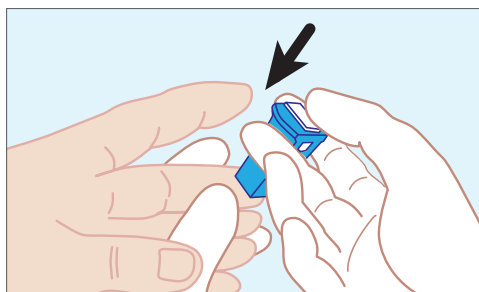

Step 3

Firmly press the white button with thumb to activate lancet. Dispose of lancet in appropriate sharps container.

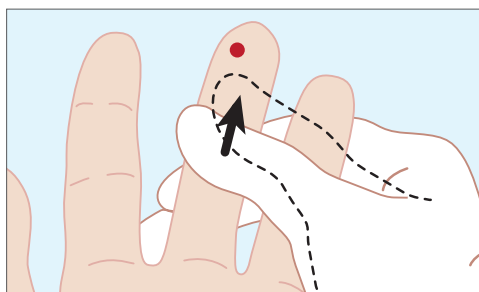

Step 4

Gently massage from the hand to near the puncture site to obtain the required blood volume.

Refer to the product labels and package insert for complete warnings, precautions, potential complications, and instructions for use.

Complete instructions for use are supplied with device.

RX ONLY. The advertisement is directed to physicians only, and not to consumers. Refer to product labels and packaging insert for complete warnings, precautions, potential complications, and instructions for use. Products may not have regulatory approval in all countries. Please contact your local sales representative if you have questions about the availability of products in your area.



TERUMO MEDICAL CORPORATION 265 Davidson Avenue Somerset NJ 08873 U.S.A.

TERUMO and TERUMO are trademarks owned by Terumo Corporation, Tokyo, Japan, and they are registered with the U.S. Patent & Trademark Office. CAPIJECT is a registered trademark of Terumo Medical Corporation, Somerset, NJ. © 2023 Terumo Medical Corporation. All rights reserved. All brand names are trademarks or registered trademarks of their respective owners. PM-06771

